

Wg

WHITE

Cube



DEFINITION

16% home bleaching gel.

INDICATION

Home dental whitening under the direction of the dentist.

COMPOSITION

- 16% carbamide peroxide;
- Thickener;
- Desensitizing agents;
- Neutralizing agent;
- Preservatives;
- Glycol;
- Flavouring substances;
- Deionized water.

TECHNIQUE OF USE

Before the use of the product, the user should have a consultation with a dental professional.

The product should be supplied by the dental professional together with personalised dental trays.

1. Remove the blister from the base and open it by turning the plastic cover tip;
2. Dispense 1 drop of the whitening gel on each tooth of the dental trays. In case of excess gel, remove it with a toothbrush;
3. Insert the trays with the bleaching

gel into the mouth;

4. The application time should be a maximum of 2 hours/day;

5. After time has passed, remove the trays, rinse, dry and store it in a dry place away from moisture and heat;

6. Brush teeth slowly. Patients with a history of tooth abrasion or tooth erosion should wait for 30 minutes before brushing;

7. The average whitening period is 2 weeks depending on time of use and initial tooth shade.

STORAGE

Store the product at room temperature between 5 and 25°C. Do not freeze the product. Protect it from direct light and heat.

PRECAUTIONS

- Tooth whitening should always be carried out under the supervision of a dentist;
- The patient must be in good oral hygiene and good general health conditions;
- This product is not recommended during pregnancy, breastfeeding and for children under 18;
- The patient should be instructed on

- the technique and treatment results;
- The patient should be warned that My White Cube gel does not whiten direct or indirect restorations;
 - Tooth bleaching may be ineffective in cases of structurally altered teeth (amelogenesis imperfecta and fluorosis stains) or very pronounced discolouration caused by tetracycline and other drugs taken systemically during tooth formation;
 - The patient should be warned of a possible increase in tooth sensitivity to cold stimulus during treatment. To stop the sensitivity, simply decrease the frequency (e.g., whiten every other day) or the time of use. If the sensitivity persists, stop the use of the product;
 - This product cannot be used on patients allergic to any of its components;

My Med Kft - Vendel utca, 11
1096 Budapest - Hungary
Tel. +36 1 613 0045
www.my-med.eu - info@my-med.eu